BeppyCUP

MEDICAL SILICONE MENSTRUATION CUP

USER MANUAL

EN

**Hello,
Welcome to Beppy.**

Thank you for choosing the Beppy Cup®. Please take a moment to read this user manual prior to using your cup. For a full overview of this product visit www.BEPPY.com.

Beppy Cup® is a reusable, menstrual cup made of Tulip-Thin, smooth medical grade silicone that offers you the ultimate comfort and fresh feeling during your period.

Beppy Cup® is the most advanced alternative for tampons and pads. It provides invisible protection up to 8 hours. You can use the Beppy Cup® for several years, which helps you save money. It is suitable for Sexual Relations, does not affect the Vaginal Flora and is Environmental friendly.


The information and advice contained in this manual is not meant to be offered as medical advice. If you have any gynecological or medical concerns, we advise you to consult your medical practitioner prior to using Beppy Cup®.

If you are using the Beppy Cup® for the first time, try to use it when you have time to experiment, finding your perfect fold and finding which position suits you better!

Now, lets start! Before first use, wash your hands before inserting and sterilize the Beppy Cup® for about 3-5 minutes in boiling water, or use your Beppy Steam Sterilizer™.

Follow these 5 steps to place your Beppy Cup® inside:


1. HOLD - 2. FOLD - 3. INSERT - 4. PUSH - 5. REMOVE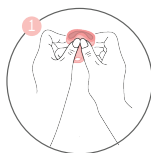

HOLD and Squeeze the Beppy Cup® in the middle with two hands. In the beginning it could take some practice to insert the cup and to find the right spot.


Sometimes the Beppy Cup® fits perfectly right away and other times you have to try a bit harder. However, time and practice will make it easier for you every time!

1. HOLD - 2. FOLD - 3. INSERT - 4. PUSH - 5. REMOVE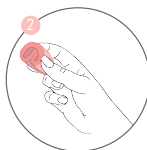

FOLD your Beppy Cup® in half, this is called the C or U fold. Keep squeezing your cup in half with two fingers, by holding your cup at the upper section. Make sure, not to hold the cup at the bottom part of the cup, because it will open before it reaches the Cervix.

1. HOLD - 2. FOLD - 3. INSERT - 4. PUSH - 5. REMOVE

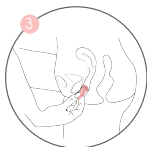INSERT your Beppy Cup® in your Vagina. Keep holding the cup from the upper section of the cup until you reach your Cervix or as far as it will go. When the entire cup is inside, simply remove your fingers and let it pop open.

1. HOLD - 2. FOLD - 3. INSERT - 4. PUSH - 5. REMOVE

PUSH your Beppy Cup® into your Vaginal Canal with your thumb or index finger. Make sure the cup is unfolded by feeling around the base of the cup. It should be flat or oval. If you feel any dents or folds on the base, it means it is not unfolded. *Gently hold the base of the cup and try to rotate it from side to side to ensure it is sealed.* When in place, try to pull the loop a bit. If you feel some resistance, a vacuum has been created and the cup has been placed successfully.

1. HOLD - 2. FOLD - 3. INSERT - 4. PUSH - 5. REMOVE

REMOVE your Beppy Cup® by gently squeezing the base to release the vacuum and slide your cup out. Gently place your finger on the loop and twist it from side to side.

Finally, empty the cup into the toilet or sink and flush it with cold water. Pay extra attention to the cleaning of the air holes and the space between the inner flap and the cup.

CLEANING AND DISINFECTING:

Since the menstrual intensity differs from woman to woman, the time between emptying the cup also varies per woman. We recommend that you empty the Beppy Cup® at similar intervals, in which you have previously changed tampons or pads. Then you get a sense of how often you need to drain your cup. To change the cup, simply empty it into the toilet or sink.

Important: it is *not* necessary to remove your Beppy Cup® when going to the toilet.

1. RINSING - 2. STERILIZING - 3. STORAGE

RINSE your cup every 8 hours during your cycle, cleanse your Beppy Cup® with cold water and a PH-neutral soap or baby whipes. While it is preferable to sterilize the cup or use a second one, if you do not have any other choice, you can also reinsert it after rinsing.

Tip: If necessary, use a small brush to clean the air holes and flap.

If you are not comfortable inserting the same cup, Beppy® provides 2 menstrual cups with every packaging so you can be clean and fresh every time!

1. RINSING - 2. STERILIZING - 3. STORAGE

STERILIZE your Beppy Cup® before using it for the first time, in between your period and after your period ends. You should cleanse your Beppy Cup® a bit more "thoroughly" in cases when your cup is not being used for a long period of time. To sterilize your Beppy Cup® you have several options:

- a. Boil your Beppy Cup® in a deep cooking pot filled with plenty of water for about 3-5 minutes. The cup does not have to be completely dry for re-insertion.
- b. Put your cup in a special microwave container with water and heat it for 3 minutes.
- c. Place your Beppy Cup® inside your Beppy Steam Sterilizer™ with a little bit of water, push the button and wait until the Sterilization cycle is completed.

1. RINSING - 2. STERILIZING - 3. STORAGE

STORE your Beppy Cup® with sufficient air supply. Make sure your cup is completely dry when storing it from one Period to the other. Pay extra attention to the cleaning of air holes and inner flap to prevent any bad smell or bacterial growth. The supplied bag and packaging are ideal for this. Don't throw it away, help the environment and reuse it as much as possible, possibilities are infinite.

TWO CUPS AGAINST TSS:

The risk of developing T.S.S. (Toxic Shock Syndrome) or other infections is higher when using menstrual products. To decrease the risk of T.S.S, Beppy® provides 2 cups instead of 1. Having a second cup allows you to store the used cup and insert a clean one every time.

Make sure you clean your used Beppy Cup® thoroughly when you get home, so you can always carry a clean cup with you. Remember to *always* change your cup every 8 hours or less.

Though many manufacturers recommend wearing menstrual cups for up to 12 hours or more, scientific research published by the 'American Society of Microbiology®' states that wearing a menstrual cup or a tampon for more than 8 hours can increase the risk to develop T.S.S.

In Beppy® we are compromised with cleanness and health, please change your cup as often as possible and before inserting the cup, make sure it is sterilized.

SEX AND THE BEPPY CUP®:

The Beppy Cup® is specially designed for women who want to have sex during their period. Beppy Cup® is the first reusable menstrual cup in Europe and Latin America. This cup can be used during sex, providing all the benefits of a cup and a mess-free period sex, thanks to its Tulip-Thin medical grade silicone and its anti leak interior flap. *It is recommended to have sex with the Beppy Cup® in your lighter days.*

The Beppy Cup® can be worn during sexual intercourse, without being felt, because it does not block the Vaginal Canal. Remember Beppy Cup® is not a contraceptive and will not protect you from STDs or pregnancy.

BEPPY CUP® PATENT AND REGISTRATION:

The Beppy Cup® and its Packaging are patented and registered to ensure their quality:

- 1. CNIPA Design Patent: ZL 202030029377.6
- 2. CNIPA Silicone Bag Design: ZAL 201830145353.X
- 3. EUIPO Design Registration: 006576674-0001

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