Beppy Steam + Sterilizer

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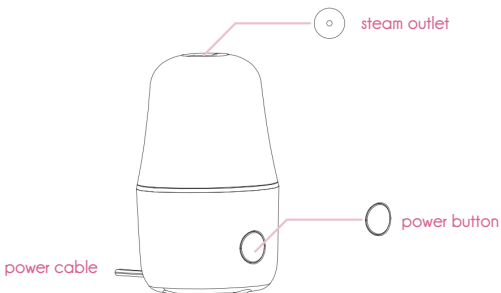
+ ENGLISH

Thank you for purchasing the Beppy Steam + Sterilizer. Please read the instruction manual for further reference before using the machine for the first time.

Safety precautions:

1. To avoid burns on the body caused by boiling water and steam, make sure to close the lid when using the sterilizer, children should stay away from the product.
2. Do not approach the steam vents when it is functioning to avoid burns.
3. Please do not place the non heat-resistant items in the sterilizer to prevent damage to the items.
4. When cleaning the product, please unplug the power, wipe with a clean soft cloth with water or alcohol, and avoid using organic solvents such as gasoline.
5. Do not disassemble or repair the product, in case of causing damage to the product.
6. After using the sterilizer, unplug the power cord and store it in a dry place.

Product Function and Parts:

Product Specifications :

Product Name: Beppy Steam + Sterilizer

Input: AC 110V / 220V

Rated power: 190W

Water line sterilization time: 6-8 minutes

Power indicator working status: Red/Pink Light

Product Features:

1. This product is specially designed for menstrual cups made of silicone, rubber or TPE.
2. Enough water capacity, long-term high-temperature disinfection, cleaning all kinds of dirt, effectively eliminating bacteria up to 99.9%.
3. Environmentally friendly made of Polypropylene (PP), high temperature resistant, corrosion resistant, double insulated design to avoid burns.
4. Built-in fuse to guarantee electric safety. Heating sheet is coated with Teflon, which is extremely durable.
5. One-button operation, heat preservation, antiburn material.
6. Stylish and beautiful appearance and convenient for travel.

How to use the product:

1. Plug the power cord of the Beppy Steam + Sterilizer into the outlet.
2. Pour a certain amount of water until it submerges the water level mark. Then put the product of your choice into the disinfection chamber and close the top cover.
3. Press the power button, the indicator light (red/pink) lights up, indicating that the machine start working.
4. Use the recommended water level mark. The sterilization time will be about 6-8 minutes. Because of the anti-dry function, the power will be automatically turned off when the water is evaporated, please turn off or unplug the power cord when the disinfection is finished.

Remarks:

Please, check your product information before steaming with the Beppy Steam + Sterilizer. The product you want to sterilize needs to hold up to 100 Degrees Celsius.

Precautions:

1. For product quality problems caused by non-human factors within one year from the date of purchase, please contact the dealer for complete machine replacement.
2. Please provide a description or picture when the quality issues involved.

Non-warranty:

1. Inadvertent use, incorrect operation and human-induced damage.
2. Cannot provide proof of purchase.
3. Damage caused by irresistible forces such as natural disasters.

Descaling:

After a while, limescale can build up in the device, because tap water contains calcium and magnesium minerals. This is especially the case when boiling or heating tap water. This is not dangerous to human health.

1. Fill the Beppy Steam + Sterilizer with a mixture of water and lemon juice or vinegar, so that all lime is under the mixture.
2. Turn on your Beppy Steam + Sterilizer and let it boil for 8 minutes. If necessary, repeat steps 1 and 2 again.
3. When you see that the Beppy Steam + Sterilizer is lime-free, repeat the process with clean water.

You have now finished descaling your Beppy Steam + Sterilizer.

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Copyrights and all rights reserved by Asha International (NL).